



# Talking Button Clock



FRONT

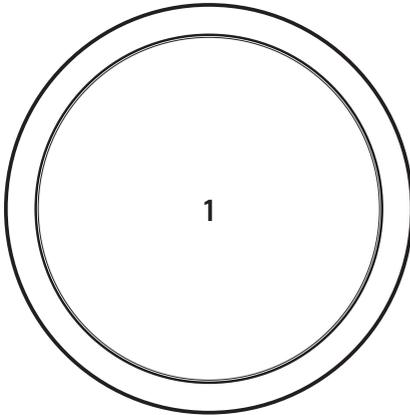

BACK

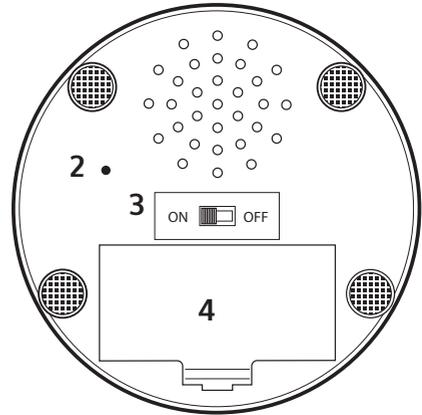

1 Main Button

3 ON/OFF Switch

2 Setting Button

4 Battery Clip

**Included:**

x2 AAA batteries

## BATTERY INSTALLATION

To install the batteries, open the BATTERY CLIP and insert the batteries (make sure the batteries are the correct way around) and replace the BATTERY CLIP.



- We recommend replacing the batteries once a year.
- Only use AAA (1.5V) batteries
- If the clock is not being used for a long period of time then remove the batteries.
- Please discard of batteries responsibly

# INSTRUCTIONS OF USE

## Settings

Locate the SETTING BUTTON on the back of the device. Successive presses of the SETTING BUTTON will take you through the settings menu which are announced in the following order:

1. Set Hour (All hours – AM/PM)
2. Set Minute (0-59)
3. Set Year (16-26)
4. Set Month (January – December)
5. Set Day (1st – 31st)
6. Set Alarm Hour (All hours – AM/PM)
7. Set Alarm Minute (0-59)
8. Set Alarm

## TO SET THE TIME

1. Press the SETTING BUTTON once, you will hear the command 'Set Hour'.
2. Press the MAIN BUTTON to select the correct hour. Once you hear the correct hour press the SETTING BUTTON and you will hear 'Set Minute'.
3. Press the MAIN BUTTON to select the correct minute.
4. Once the hour and minute is correct release all buttons for 5 seconds until you hear a short beep. This beep signifies that is has come out of the settings mode.
5. To confirm the time, press the MAIN BUTTON and the clock will announce the time you have set.

To correct or adjust the time again, press the SETTING BUTTON until you hear the setting mode you want to adjust and follow the above steps to adjust the time.

## TO SET THE DATE

1. Press and release the SETTING BUTTON until you hear the command 'Set Year'.
2. Press the MAIN BUTTON to select the correct year, this will be heard as numbers 16-26 (2016-2026). Once you hear the correct year press the setting button and you will hear 'Set Month'
3. Press the MAIN BUTTON to select the correct month. Once you hear the correct month press the SETTING BUTTON and you will hear 'Set Day'
4. Press the MAIN BUTTON to select the correct day. Once you hear the correct day release all buttons for 5 seconds until you hear a short beep. This beep signifies that is has come out of the settings mode.
5. To confirm the date you have set, press the MAIN BUTTON once and you will hear the time, press twice and you will hear the date announced.

To correct or adjust the date again, press the SETTING BUTTON until you hear the setting mode you want to adjust and follow the above steps to adjust the time.

### **VOICE SLEEP MODE FUNCTION**

The sleep mode function can be used to silence the clock so that no sounds will work when pressed.

#### **To activate the sleep mode:**

Press and hold the MAIN BUTTON for 8 seconds, you will hear 2 short beeps which confirms that the clock is now in sleep mode.

#### **To deactivate sleep mode:**

Press and hold the MAIN BUTTON for 8 seconds, you will hear 4 short beeps which confirms that the clock voice has now been reactivated.



Using the ON/OFF switch to deactivate the clock will reset all settings.

## TO SET THE ALARM

1. Press and release the SETTING BUTTON until you hear the command 'Set Alarm Hour'.
2. Press the main button to select the correct hour. Once you hear the correct hour press the SETTING BUTTON and you will hear 'Set Alarm Minute'.
3. Press the main button to select the correct minute.
4. Once the desired alarm time is set, press the SETTING BUTTON to hear 'Set Alarm'
5. Press the MAIN BUTTON to hear 'Alarm On' or 'Alarm Off'  
ALARM ON is followed by a high pitch double beep  
ALARM OFF is followed by a low pitch double beep  
The clock will make a high pitched double beep sound to indicate the alarm is sounding. Turn the alarm off by simply pressing the MAIN BUTTON, this will deactivate the alarm and announce the time .

## MAINTENANCE

Do not use any chemical solutions or corrosives to clean the clock. A soft cloth can be used to keep your clock clean.

## TERMS & CONDITIONS

12 months manufacturing guarantee from the date of purchase, if you have any issues with the product then please contact your retailer.



This product is CE marked and fully complies with all applicable EU legislation.