



You should have these parts:

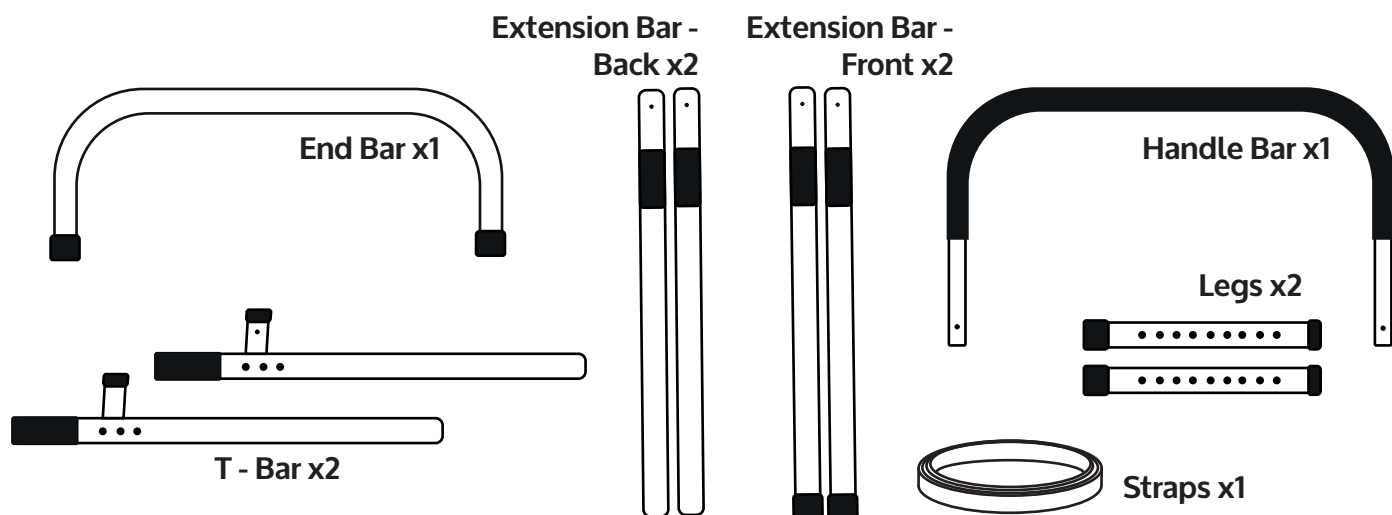

Verify all parts are included before assembly.


Instructions

1. Insert Extension Bar-Front into Extension Bar-Back and hand-tighten the threaded connectors. Set aside.

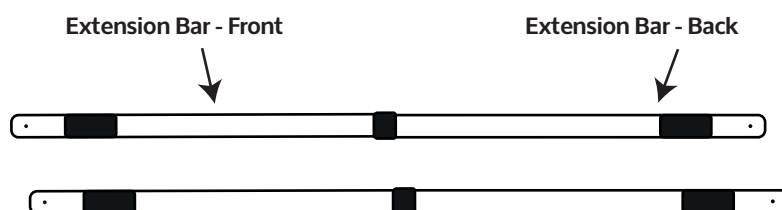


2. Insert T-Bars into each side of the Handle Bar. You may need to loosen the ridged clamps a little and tighten after inserting the Handle Bar.

Lay the assembled part on a flat surface.

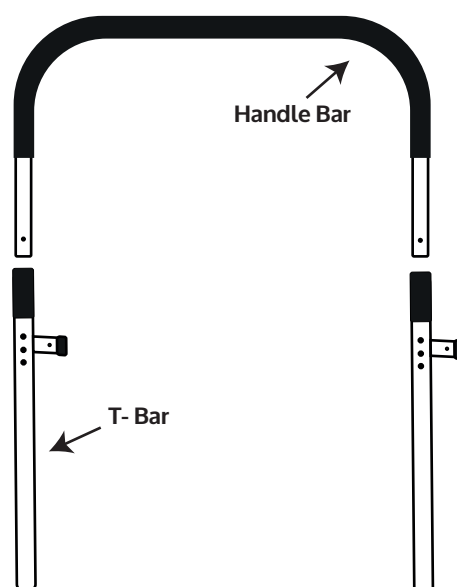


Accenture Bed Rail with Floor Support - BD06026

- 3.**
Take the assembled Extension Bars and insert the larger diameter into the T-Bar horizontally.

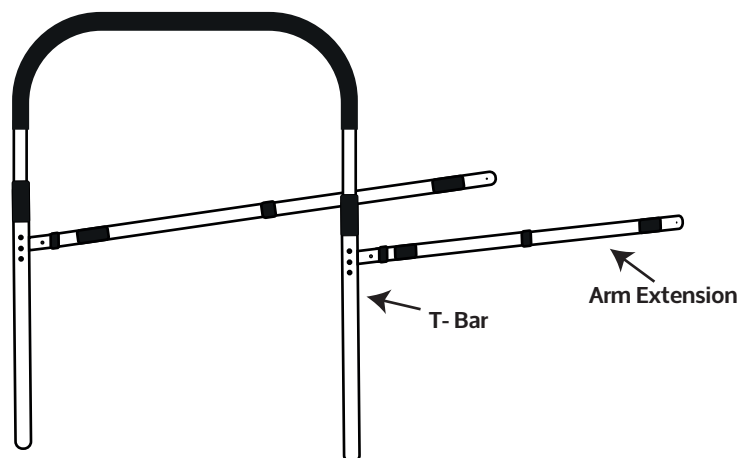


- 4.**
Take the End Bar and insert it into the end of the Extension Bar assembly.

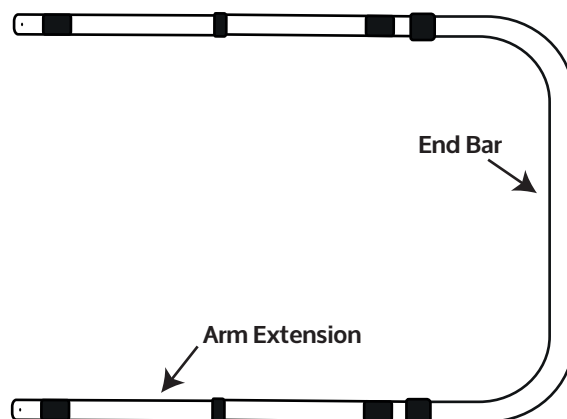

- 5.**
Insert the Legs into the T-Bar and adjust to the required height using the push-pin buttons.

- 6.**
Adjust the length of the Extension Bars as needed and tighten all threaded connectors. Place the strap around the mattress and fasten.

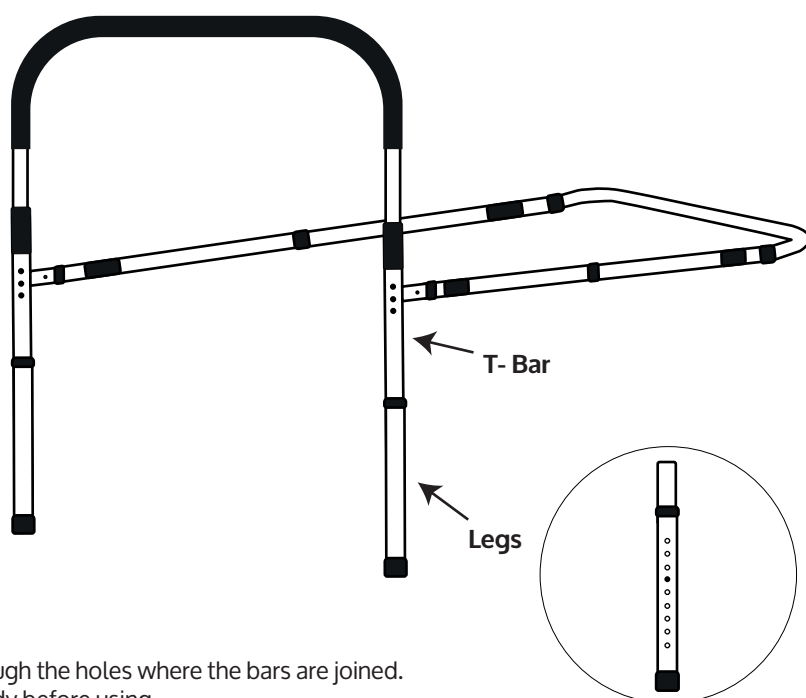

Warning!

Before use, check that all push-pin buttons fully protrude through the holes where the bars are joined.
Ensure both legs touch the floor and that the hand grip is steady before using.