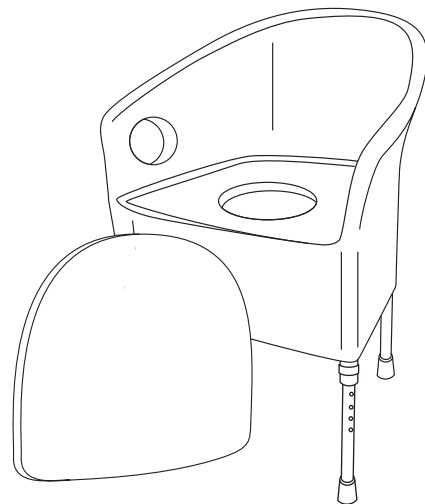
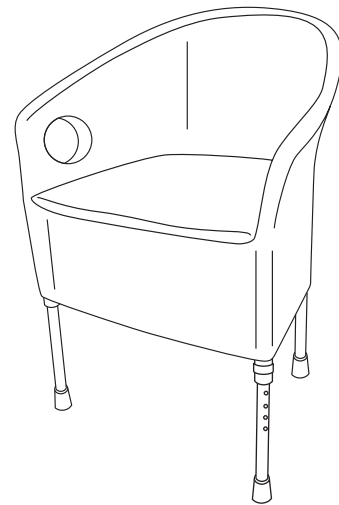


RATTAN ADJUSTABLE COMMUNE

Welcome to your new Albany Rattan Adjustable Commode. A height adjustable commode with a rattan style weave making it a discreet commode ideal for home use.

USING YOUR COMMUNE

- Insert the adjustable legs with the clips and set them to a suitable height.
- To remove the pan, remove the seat padding and internal seat cushion.
- Before use, make sure the pan lid is removed and the internal seat cushion is securely in place.
- After use, remove the internal seat cushion, replace the pan lid and use the carry handle to dispose of the contents.
- Clean the seating area after each use.
- Ensure each leg is set to the same height and the clips are secured in place.
- Check the commode is stable before each use.
- Regularly check that all fixings are tight.

**TO CLEAN YOUR COMMUNE**

- Wipe clean using soapy water and a non-abrasive cloth.

**Max user weight: 190kg**