


AREZZO

LUXURY BATH STEP

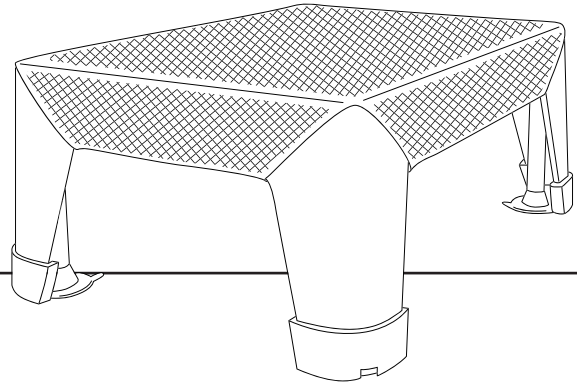

PRODUCT CODE: BA03032

The Arezzo bath step is designed to be used as an aid for getting in and out of the bath.

Features:

- Stable frame
- Non-slip suction feet
- Non-slip cover

Releasing the suction feet:


To release the bath step from the floor, use the pull tag (A) on each leg to release the air and easily lift the bath step.

Removing the grip mat:

To remove the grip mat pull the plug fasteners that sit on the underside of the step away from the plastic frame. When reattaching the grip mat please ensure all plug fasteners are firmly in place before using.

Important:

- Do not use if defective
- Do not make any unauthorized alterations or modifications to the product
- Do not try to move the product when in use.
- Before every use check the product for damage and ensure that it is securely assembled.
- Do not exceed the maximum weight limit

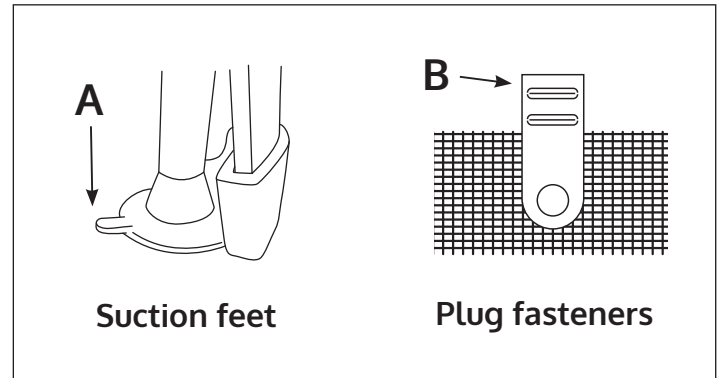

Specifications:

① Height: 173mm	③ Length: 418mm
② Width: 316mm	Product Weight: 1.832kg

MAX USER WEIGHT: 150kg / 24 stone

Maintenance:

To clean the product, remove the grip mat and clean by hand.

Warranty: 3 years