



Thank you for purchasing the Bathboard and Swivel Seat from CareCo, we hope you are pleased with your purchase.

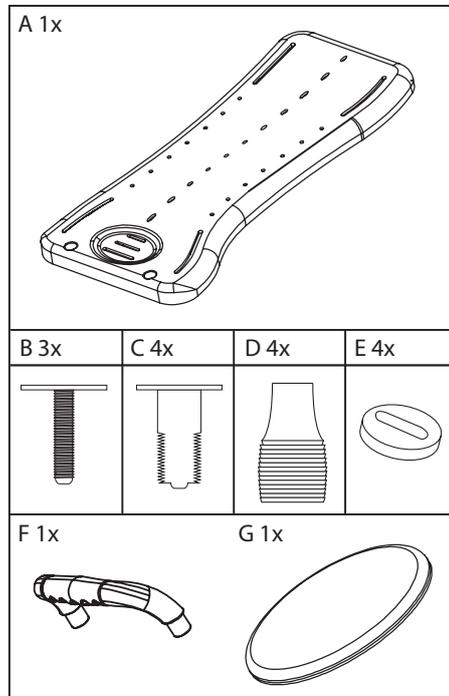


- A 1x Bathboard
- B 3x Plastic Bolt (Circular)
- C 4x Plastic Bolt (Rectangular)
- D 4x Bath board feet
- E 4x Plastic Washer
- F 1x Transfer Handle
- G 1x Swivel Seat

ASSEMBLY


1. Ensure all parts are present and remove all packaging
2. Push the red transfer handle into the two holes at the end of the bathboard with the handle angled towards the bathboard.

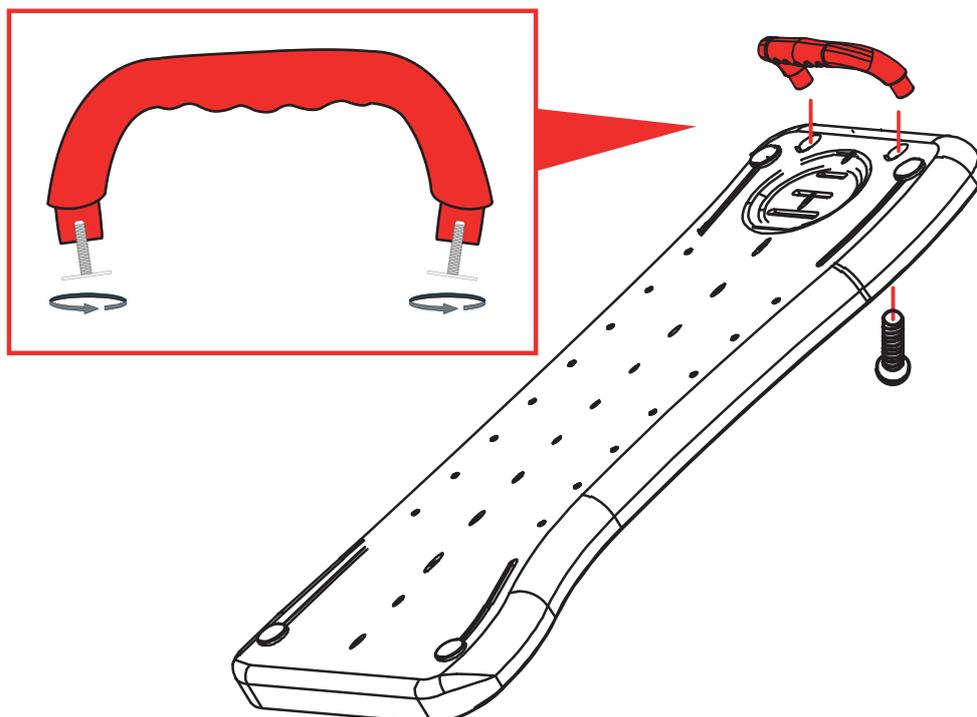

3. Using the rectangular plastic bolts and washers, secure the four feet to the underside of the bathboard. The bolt goes through the surface of the bathboard, through a washer on the underside and then screws into the feet.

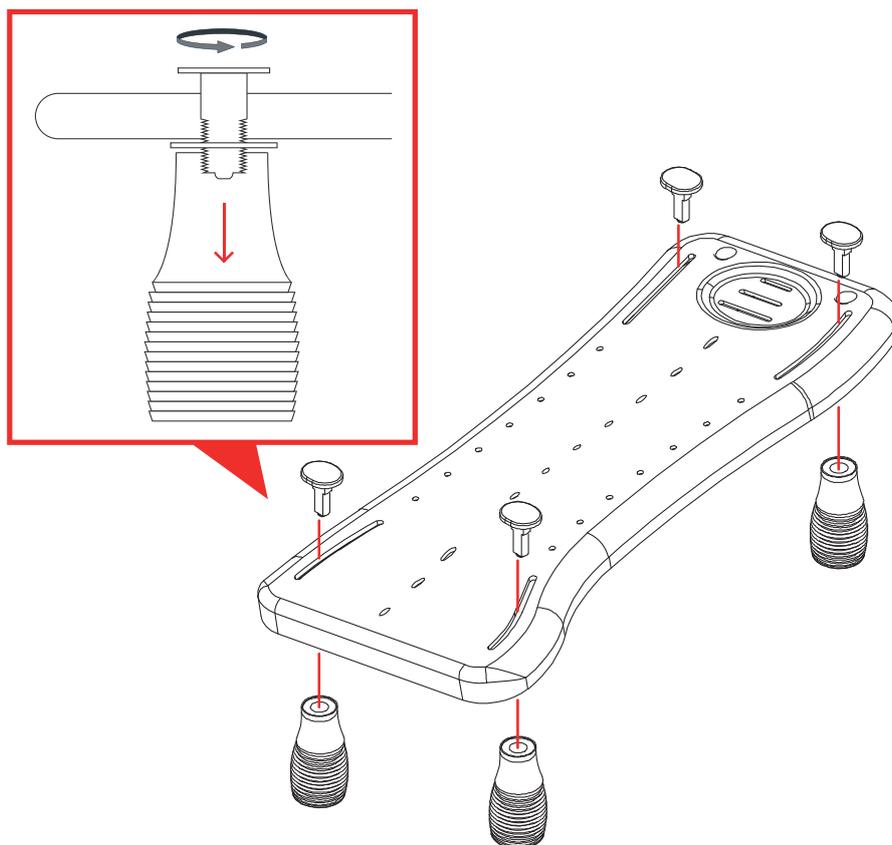


4. Place the blue swivel disc into the hole in the middle of the bathboard. Whilst holding in place, use the circular bolt to secure in place from the underside. Tighten until secured in place but can still rotate. Do not over tighten as the disc will not swivel

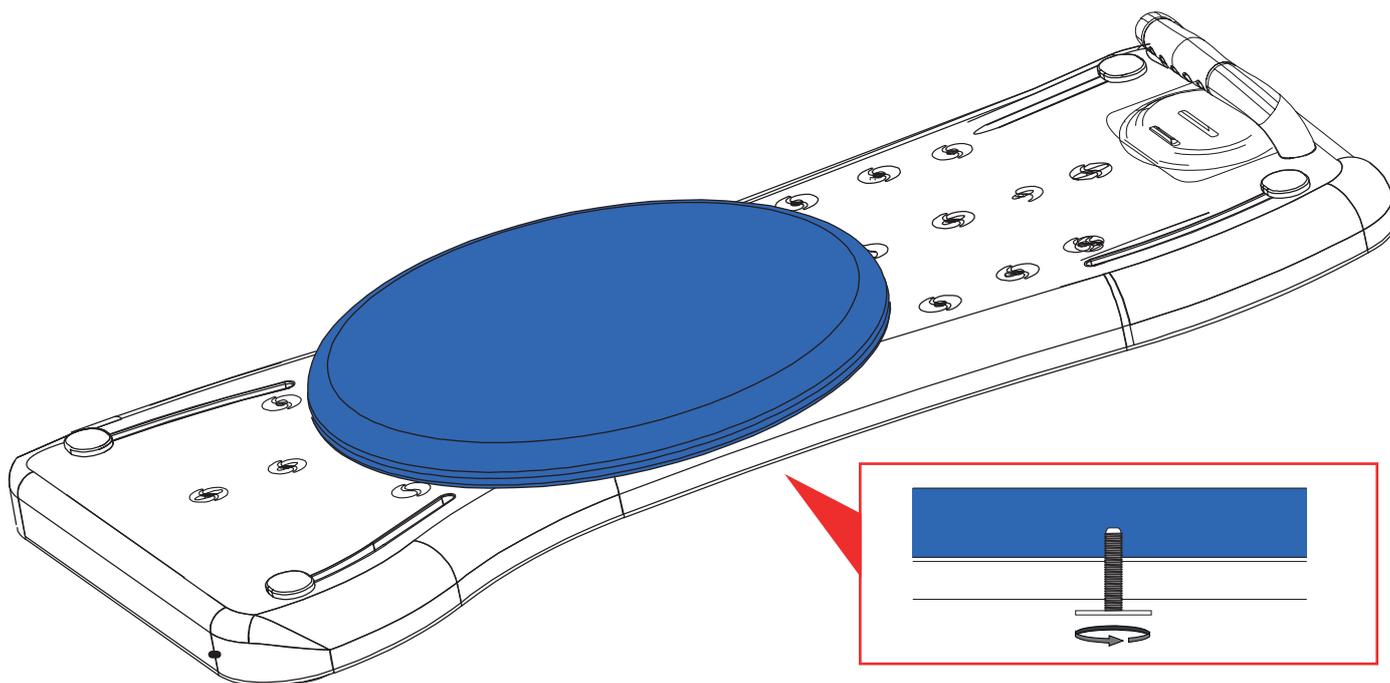

Operation

1. Secure the bathboard in the bathtub using the width adjustable feet on the underside of the bathboard. If necessary, loosen the feet and retighten in desired position.

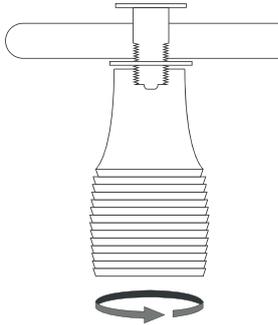

2. Place bottom in the centre of the swivel disc, raise legs up, and rotate until square in the bath.

Please Note

Be sure your Bathboard and Swivel Seat is in proper working order at all times. Please contact CareCo for any repairs required.

- Bathboards are used accross a bath width ways.
- Store in a dry and ventilated place.
- Wipe down the frame using a soft cloth at least once a week.
- Clean the seat upholstery using a mild soap solution, rinse and wipe dry.
- Do not use this product if defective.
- Observe the alignment; ensure you are happy with the fit to the bath.
- Do not move under load i.e. with the person on Bathboard.
- If the user has limited mobility, please do not leave unattended for safety purposes.
- Max user weight of 150 kg (23.5 stone).

Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the wheelchair or provide parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors, grips and arm pads, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.



**CareCo, Hubert Road, Brentwood,
Essex CM14 4JE
www.careco.co.uk**