




Product Code: MS05016


Congratulations on purchasing the CareCo Mobility Store that is designed to keep your mobility scooter clean and safe.

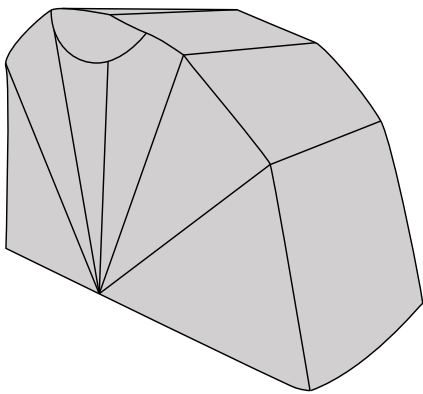

The CareCo Mobility Store comes partially assembled, so it takes only minutes to complete.


Start by checking the various components listed in the right chart to familiarise yourself and make sure that you have all of them.

### Components

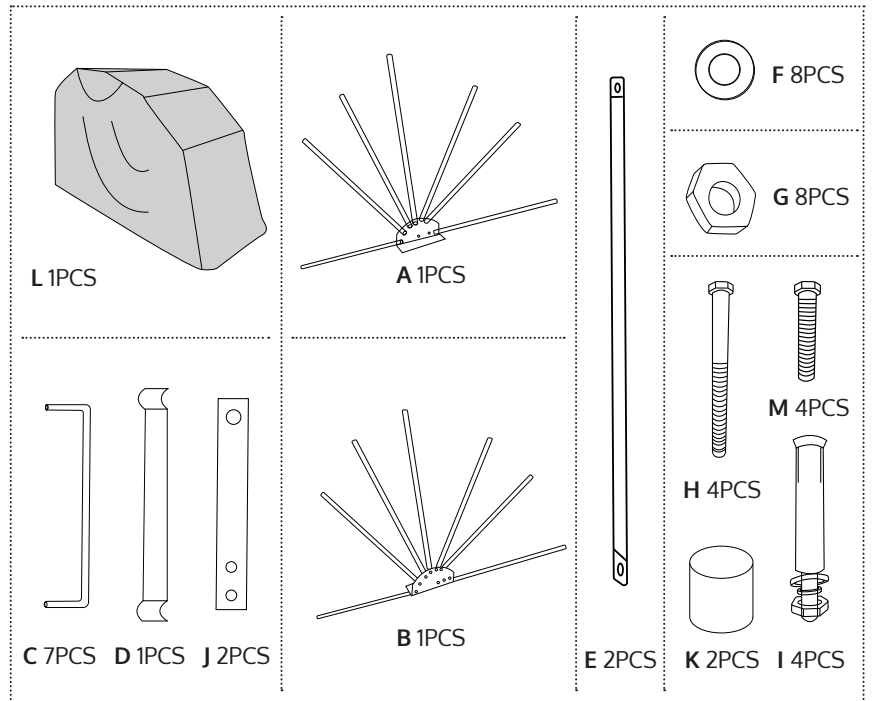



### Features

- 1 - Inside storage pockets
- 2 - Ventilation flaps
- 3 - Charging Windows

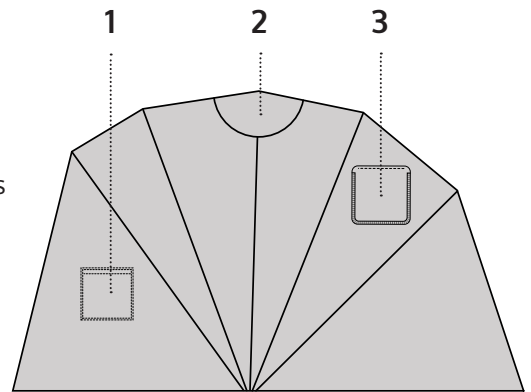

### Step 1

Lay assembly A on your right and B on your left with the shortest tubes pointing in front of you and ribs laying fanned out on the floor.

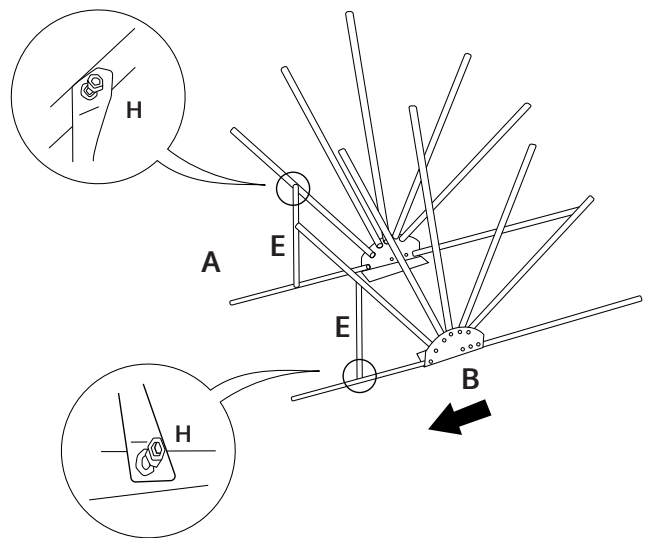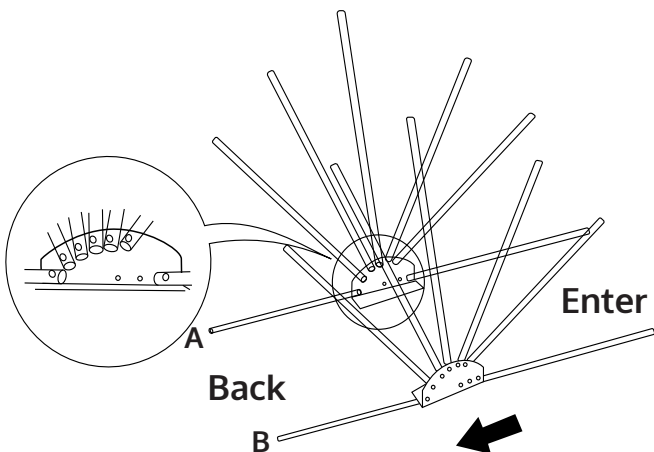



### Step 2

Connect the E tubes on to the A & B parts with angled ends on upper rib using F, G & H fasteners.

### Step 3

Connect D. Then lift the assemblies with all ribs and connect C across the A & B frames.

Leave the last C tube uninstalled for this step as the picture shows.

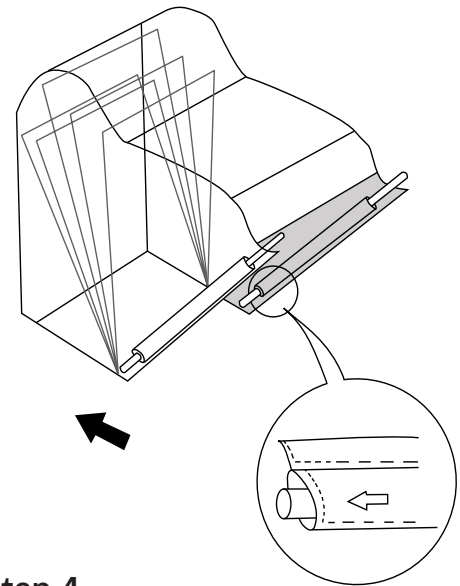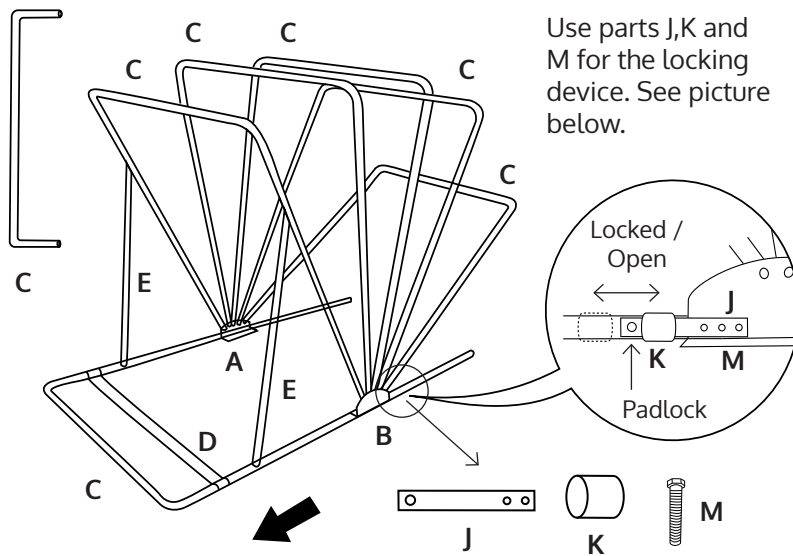



### Step 4

Check to ensure that the tent cover is spread in the right direction.

Let the 2 unconnected tubes pass through the ground sleeves of the cover.

### Step 5

Let the last C tube pass through the ground sleeve of the cover and connect it with assembly A and B.

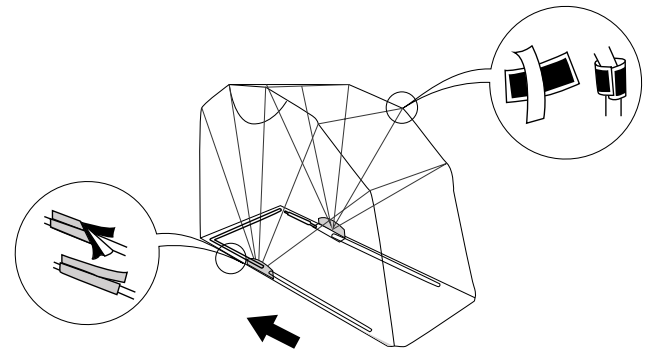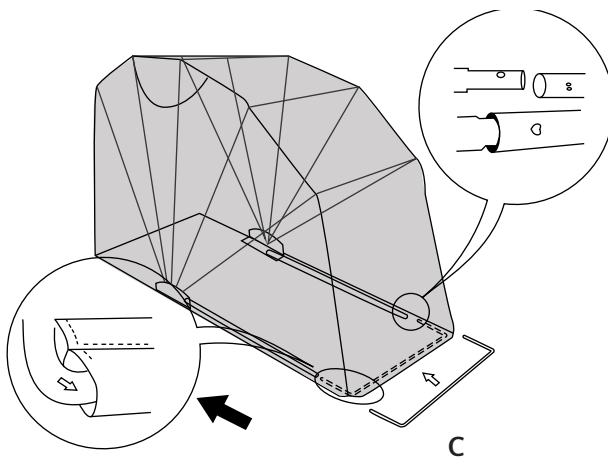

### Step 6

Fold the cover backwards over the frame, wrap the velcros around the rib as appropriate, and tidy up the whole tent.

#### Secure the frame

Weights can be placed on the flat bar connected to the D plate. Bolts can be used to fasten the swivel plates of A & B to concrete or other firm surfaces.

#### Fold down and storage

The CareCo Mobility Store is designed to fold down for storage as needed. While the Mobility Store is still fully assembled, remove the bolts for the A frame and take out the H tubes. The Motor tent can then collapse closed pivoting on the A and B base assemblies. Tie the ribs together at the top and then you can easily move it around.

#### One Year Limited Warranty

For one (1) year from the date of delivery, we will replace at our discretion to the original purchaser any part found faulty upon examination by an authorized representative of CareCo. CareCo are not liable for damage caused by an act of god, for example a heavy thunderstorm or gale force winds.