



Secco 4

Rollator

Product Code: WA01073

OWNERS HANDBOOK

www.careco.co.uk

Your CareCo Secco4 Rollator



List of Parts

1. Brake Handle/Lever
2. Backrest
3. Tightening Handle
4. Seat
5. Underseat Bag
6. Rear Wheel
7. Front Castor

Introduction

Thank you for purchasing the CareCo Secco4 Rollator. The Secco4 Rollator has been designed and manufactured to meet our own high level of specifications and we hope you will be delighted with your purchase.

The Secco4 Rollator is a portable rollator to aid the user to walk and offer stability and security. The Secco4 is designed for use both indoor and outdoor and will take a maximum user weight of 110kg (17 stone).

Specification

Rollator Dimensions: 62cm (Length) x 62cm (Width) x 78-93cm (Height)

Folded Dimensions: 20cm (Length) x 62cm (Width) x 81cm (Height)

Seat Height: 53cm

Weight: 6.6kg

Max User Weight: 110kg

Assembly

The CareCo Secco4 Rollator will arrive assembled with the exception of the height adjustable handles.

1. Remove all protective packaging and fold out the rollator. Place upright with wheels on the ground on a flat surface.
2. Ensure the star knob and bolt are removed from the frame (they may or may not arrive this way). (fig. A)
3. Insert the handle tube into the rollator frame ensuring the cable is on the outside of the rollator frame.
4. Select the required height and insert bolt through the frame and handle, making sure the hexagonal head on the bolt sits in it's recess. (figs. B & C)
5. Attach the star knob to the other side onto the bolt and tighten. (fig. D)



6. Repeat for the other handle.
7. Adjust the height of the handles, reverse the above process.

For Safety:

- Ensure you adopt a comfortable posture and can operate the brakes with ease.
- Ensure the handles are at equal heights and securely locked in position.
- Ensure the rollator is fully folded out prior to use.

- Ensure the brakes are tested prior to use.
- Do not use on gradients greater than 7 degrees.
- Ensure the brakes are locked when using the seat.
- When sitting, ensure the user is sat in the centre of the seat

Operation



1. To operate the brakes squeeze the lever up to engage the brake
2. To lock the brakes, push down on the brake lever until it locks in place
3. To unlock the brakes, pull up on the brake lever until the lock is removed

Folding

For ease of storage and transportation the CareCo Secco4 has an easy fold mechanism.

To fold the Secco4, lift the seat up and pull the black cord whilst holding the rollator handles, lifting the rollator at the same time. This will fold the rollator for easy storage and transportation.

Maintenance

Be sure your CareCo Secco4 Rollator is in proper working order at all times. Please contact CareCo for any repairs required.

- Ensure the walker is kept clean. Wipe down the walker with a damp cloth on a regular basis.
- Check the wheels and height adjustable handles regularly.
- Ensure the brakes have been adjusted to suit the wheels and user.
- Ensure all nuts, bolts and screws are securely tightened at regular intervals.
- If the solid wheel becomes cracked or severely worn, replace immediately
- Check the front castor wheel swivels freely.

Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the Rollator or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications



**CareCo, Hubert Road, Brentwood,
Essex CM14 4JE**

www.careco.co.uk



CareCo
the discount mobility experts