

# Foot Warmer - BL06139

## Instruction Manual



### Thank You...

Thank you for purchasing the CareCo Foot Warmer. The Foot Warmer has been designed and manufactured to meet our own high level of specifications and we hope you will be delighted with your purchase. Please read the following operating and maintenance instructions carefully.

### Safety Information

This instruction manual belongs to this device. It contains important information about setup and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device. Please Note, warranty is not transferable.



Do not wash the foot warmer. The cover can be washed in a normal was at 30°C maximum.




Do not use bleach on Foot Warmer.



Do not tumble dry.



Not to be used by children under 3 years old.

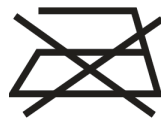

Do not iron.



Do not dry clean.




Do not use Foot Warmer when it is folded.



Do not puncture the Foot Warmer.



Only use the product indoors.

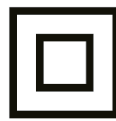

Protection Category II.

## Safety Information

- Carefully check the foot warmer before each use for signs of wear and/or damage.
- Do not use if you notice wear, damage or signs of improper use on the foot warmer, the switch or the cables. Contact CareCo if you notice any signs of this.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- The appliance may not be switched on using a clock timer.
- The foot warmer may not be operated while folded or creased.
- No safety pins or other pointed or sharp objects may be attached to the foot warmer or stuck into it.
- The unit is designed for private home use only.
- This foot warmer is not intended for clinical or commercial use.
- Only use the foot warmer for the purpose described in this instruction manual.
- Do not use the device on children or persons who are insensitive to heat. (Persons who cannot react to overheating.)
- Children below the age of 3 years are not allowed to use this device, because they are not able to react to overheating.
- This appliance can be used by children aged from 8 years and above, persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- The appliance is not to be used for warming animals.
- Never leave the foot warmer unattended during operation.
- Do not fall asleep while the foot warmer is switched on.
- Do not wear shoes when using the foot warmer because this will lower your heat-sensitivity and will cause the foot warmer to become dirty.
- Prolonged use of the foot warmer at a high setting could lead to skin burns.
- Do not use the foot warmer in bed.
- Do not use the foot warmer when it is damp or in moist environments. The foot warmer may only be used again once it is completely dry.
- The controls and leads must not be exposed to any kind of moisture.
- Do not reach to retrieve a foot warmer that has fallen into water. Unplug the appliance immediately.
- During operation, the switch and lead may not be placed on or under the foot warmer or covered in any other way whatsoever.
- Do not place any heat sources (e.g hot-water bottles or heating pads) on the foot warmer.
- Keep the mains lead away from hot surfaces.
- Do not place any large or heavy objects (e.g cases, etc.) on the foot warmer regardless of whether the appliance is switched on or off.
- Never carry, pull or turn the foot warmer using the power cable and do not clamp the cable.
- When the appliance is not in use, disconnect the plug from the mains.
- In the event of a defect, do not repair the foot warmer yourself. Repairs must only be carried out by CareCo or other suitably qualified personnel.
- A damaged power supply cable cannot be replaced. If the power supply cable is damaged, the whole foot warmer must be disposed of.
- When storing the foot warmer, leave it to cool down first before folding. To prevent creases, do not place any objects on the foot warmer when storing.
- If you have any health concerns, consult your doctor before using the foot warmer.
- Do not use the foot warmer if your feet are swollen, inflamed or injured.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

## Unpacking

Please check that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact CareCo.

The following parts are included: 1 x foot warmer with insert cover, 1 x instruction manual.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact CareCo without delay.



**Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!**

## Operation

Always make sure that the foot warmer is not creased or folded while it is in use. You must always use the foot warmer with the cover.

Switch the appliance on with the sliding switch on the control unit. Never use the clock timer!

**Switch Level 0** = Appliance switched off

**Switch Level 1** = Heat Low

**Switch Level 2** = Heat Medium

**Switch Level 3** = Heat High

During switch level 1,2 and 3 the indicator light is illuminated, showing the adjusted level number contemporaneously. You can now enjoy the pleasant warmth at your feet.

After approx. 90 minutes the foot warmer automatically switches off. To switch the foot warmer on again, you must first move the sliding switch to switch level 0. After approx. 5 seconds you can switch the foot warmer back on by sliding the switch to level 1,2 or 3.

You can stop using the appliance at any time by sliding the switch to level 0.

Disconnect the plug from the mains socket when you are not using the foot warmer.



## WARNING

When the foot warmer is in operation, the control unit may warm up slightly. For this reason, the control unit must not be covered or placed on the foot warmer whilst the appliance is switched on.

Do not rely on your hands to test the heat of the foot warmer. Hands are considerably less sensitive towards heat than the rest of the body's surface. A temperature which is judged by the hand to be slightly warm can be felt by the feet to be unpleasantly hot.

The indicator light goes out if the safety system switches off the foot warmer due to a fault, even if the foot warmer is still switched on. The foot warmer must no longer be used after a fault has occurred. This is because the safe operation of the appliance can no longer be guaranteed.

## Cleaning & Maintenance

### Foot Warmer

The electric foot warmer should be kept clean because it may not be washed or dry-cleaned. Before cleaning the foot warmer, always disconnect the plug from the mains and allow it to cool down. The foot warmer can be cleaned dry with a soft brush. Small marks can be removed with a cloth or a slightly damp sponge, using a mild liquid cleaner. The foot warmer may not become wet. If it becomes damp it must be air-dried. Store the foot warmer in a dry place and do not place any objects on top of it.

### Insert Cover (interior)

Before cleaning the foot warmer, the insert cover must be taken out. The cover can be cleaned with a damp cloth or according to the care instructions on the labels. Use solvent-free cleaners only. Only use the foot warmer again once it and the cover are completely dry.

## Specifications

Name and Model	: Foot Warmer DL06139
Power Supply	: 220-240 V AC 50Hz
Heat Output	: max. 100 W
Automatic Switch Off	: after approx. 90 min
Switching Levels	: 0 - 1 - 2 - 3
Dimensions	: approx. 30(l) x 30(w) x 23(h) cm
Article Number	: DL06139

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

### Disposal



This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your local council for more information about disposal.



**CareCo**  
the discount mobility experts

CareCo, Hubert Road, Brentwood,  
Essex, CM14 4JE

[www.careco.co.uk](http://www.careco.co.uk)