

## Intended Use

Thank you for purchasing the Livewell Hearing Aid. If you find it's difficult to hearing very well form others, a hearing aid can help you regain your hearing so you can live the life you're used to.

Wearing a hearing aid means rediscovering the pleasure of gathering with friends and family, feeling more confident at work, and enjoying movies and TV programs.

## Instruction diagram



'N' means 'Power On'

'O' means 'Power Off'

'+' Positive side of Battery



'-' Negative side of Battery

'1-4' Volume grade, '1' Min, '4' Max

DL10015  
Battery: AG13 & A675



## Usage

1. Chose appropriate size ear plug and attached to the hearing aid.
2. Open the battery door in the direction shown, insert the battery as indicated by the polarity symbols in the compartment, close the battery door.
3. Attach the hearing aid onto your ear by placing the ear plug in the ear & positioning the ear piece comfortably over the ear.

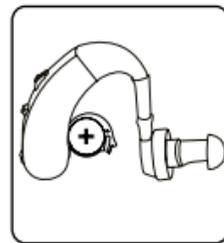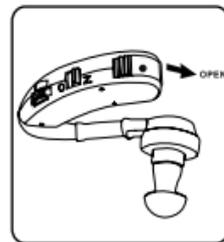

4. Turn ON the hearing aid, adjust the volume (4-level selection: 1-4) according to your personal hearing need.
5. Turn OFF the hearing aid when not in use by adjusting volume to minimum or opening the battery door to the lock hinge

**CAUTION:** If the unit will not be used for along period of time, remove the battery from the unit.

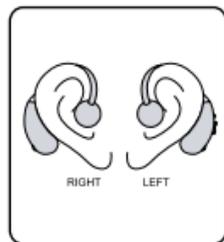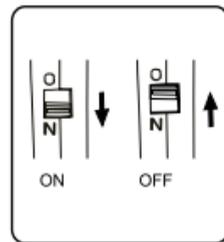

## Specification

1. Max sound output: 130±5 dB
2. Sound gain: ≥ 50dB
3. Total harmonic wave distortion: ≤5%
4. Frequency range: 300-4000Hz
5. Input noise: ≤30dB
6. Voltage: d.c.11.5V
7. Current: ≤4mA

## Maintenance

*No sound from hearing aid. Check as follows:*

1. Power is off.
2. The battery is in correct direction.
3. Hearing aid has run out of battery.
4. The earphone is damaged.
5. The wire is broken.

*Sound from the hearing aid is very weak and fuzzy.*

1. Low Battery.
2. Defective earphone.

**Use a clean dry cloth and special detergent to wipe the hearing aid.**

## Note

1. Please don't use the hearing aid at high volume for a long time. It will damage your hearing.
2. Please don't use when showering, swimming, raining or in a wet environment, **please don't expose the hearing aid under sunshine**, get close to a stove or other heat sources.

3. Please maintain the product whole all the time in order to avoid the earplug and microphone opening being blocked by foreign objects.
4. Please protect the hearing aid carefully and avoid dropping it on the floor.
5. Please don't leave the old batteries in the hearing aid for a long time as this might cause battery erosion.
6. Please dispose of old batteries in environmentally friendly ways.
7. Keep the hearing aid away from children.

## Warranty

This product has a 1 year warranty excluding batteries. Spare batteries can be purchased from your supplier.



**Livewell**  
HEARING AID

**USER MANUAL**

Product Code:  
**DL10014**

