

ADDITIONAL WARNINGS: Installation and Operation



Always ensure the backrest is connected to the base correctly.

Two Blue dots must always be shown. Never operate the bath lift if only 1 Blue dot is shown.



Ensure the backrest is always fully reclined in the lowest position before connecting the handset.



Always ensure that the bath lift runs freely (no obstructions) every time you use it.

IMPORTANT

1. Please ensure the connection from the hand control to the actuator is pushed in firmly every time you use your bath lift. Also push hand control firmly into charger when charging - this connection is really important.
2. Please ensure backrest actuator is fully retracted before taking the bath lift out of the bath.
3. Lift rear suckers first (not front) if taking bath lift out of the bath.
4. Do not fold backrest flat in the bath if there is still water in the bath.
5. Always ensure your bath lift runs freely (no obstructions) every time you use it.
6. Fully charge your handset regularly (at least once per week).

FAILURE TO FOLLOW SAFETY INSTRUCTIONS WILL VOID YOUR WARRANTY.