


Products:



BA08057
Modena Straight
Stainless Steel Bar



BA08056
Modena V Shaped
Stainless Steel Bar

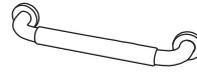

BA08055
Modena Curved
Stainless Steel Bar

Included:







SCREWS x6
(x9 for BA08056)



WALL PLUGS x6
(x9 for BA08056)

Installation Instructions:

1. Unscrew the screw cover and position the bar where you would like to install it and mark the 4 screw positions.
2. Use the screws provided along with suitable wall plugs and screw the bar securely into the wall. If fitting the Modena Stainless Steel Bar to a hollow wall, please ensure suitable fixings are used.
3. Once the bar is in place, simply tighten the screw cover to hide and protect the screw heads.
4. To remove the bar, unscrew the screw cover to access the screw heads.

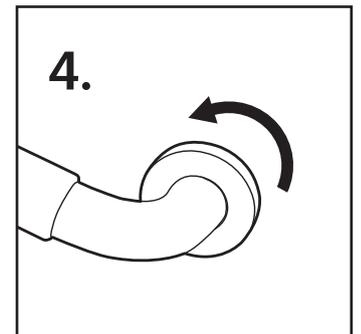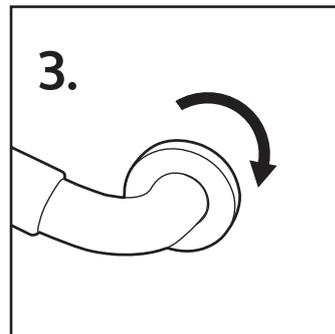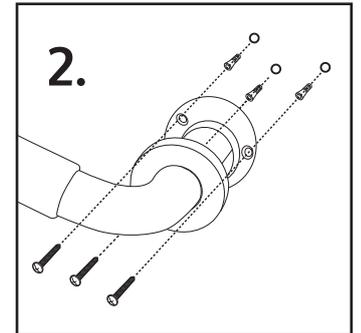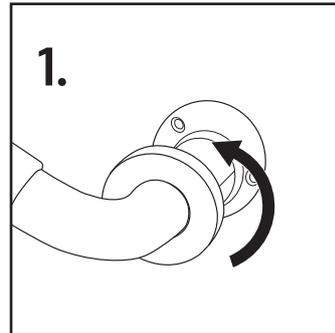

Max Load:
120 KG

Maintenance:

Regularly check that your grab bar is securely attached and the fittings have not come loose. Clean your grab rail regularly to preserve the grip and prevent germs from spreading.

Warranty: Your Modena Stainless Steel Bar is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products. In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts. If you have any queries please contact CareCo. The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

CareCo

CareCo UK Limited ,
Hubert Road,
Brentwood, Essex,
CM14 4JE

0333 015 5000