

Soho Deluxe Folding Shower Seat - BA06044

All Shower Seats

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the shower seat. Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

Cleaning

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a nonabrasive cleaner or mild detergent with a soft cloth, avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair. Certain products have clip on/clip off backs and/or seats; you can easily remove these if required, clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/or back as this will permanently damage the plastic. N.B. Regular daily cleaning will reduce the build-up of limescale on your product. UNDER NO CIRCUMSTANCES use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.



Use of Bath Oils


Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance. In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.

Care and Maintenance

Please make a safety check of the product at regular intervals (at least every 6 months) or on reissue, or if you have any concerns.

N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user

Max user weight - 190kg (30 Stone)
DO NOT EXCEED THE WEIGHT LIMIT STATED.

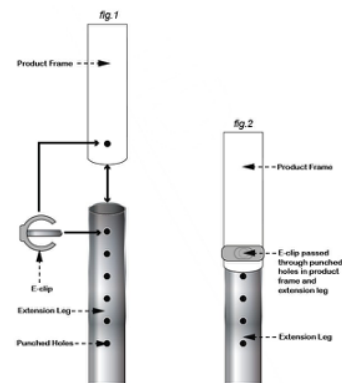

Adjustments to Extensions Legs (If Applicable)

Please adjust the legs prior to installation. Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1) Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2). Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

Fixing the Shower Seat

Choose a suitable fixing for the substrate that you intend to fix the shower seat to and secure the back plate of the shower seat to the substrate. Make sure that the fixing is secure before allowing it to be used.

Soho Deluxe Folding Shower Seat - BA06044

Testing the Shower Seat

Once you have installed the shower seat check your installation and the movement of the shower seat by raising the seat from the down position to the up position.

- Check that there are no obstructions
- Check that the movement of the seat is smooth and unhindered
- Check that the seat legs (if applicable) are sitting evenly on the floor

Raising and Lowering Shower Seat

To raise your seat, simply lift the front of the seat and the legs will automatically fold inwards towards the wall. Reverse the action to lower.

Folding Action

If required, your shower seat can be folded up when not in use. The seat is held in position by friction only. Once you have installed the seat, check that there is sufficient friction to keep the seat in the upright position when required. If the seat starts to descend, then you will need to tighten the bolts either side of the seat hinges. This operation should be sufficient to maintain the seat in the closed position. Additionally, it will be necessary for the bolts to be tightened periodically, if the seat starts to open on its own accord.

Threaded Leg Adjustment

Your shower seat may have special threaded leg adjustment as opposed to the standard extension leg. To adjust the height of your seat, carefully open the stainless steel M10 locking nut and undo the threaded stem to the desired height. On no account exceed a finished height of 470 mm measured from the top of the seat to the bottom of the black circular foot, as this may compromise the safety of the user. Once you have reached the desired height, lock the stem in place by tightening the lock nut against the black indented bush. NB. Occasionally the extension stems may be slightly angled inwards to the frame. This is not a fault and will not impair the function or safety of the product, provided that you tighten off the lock nut as required.

Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support. When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

Important Information

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement. If you have any queries please contact CareCo.