

Vista8

Wheelchair

USER MANUAL

PRODUCT CODE **WC01069**

CareCo

CONTENTS

1. SPECIFICATIONS	1
2. WHEELCHAIR COMPONENTS	2
3. INSTRUCTIONS	3
3.1 Folding and Opening	
3.2 Adjusting the Footrest Length	
3.3 Removing & Replacing Footrests	
3.4 Applying the Brakes	
3.5 Removing & Replacing the wheel	
4. SAFETY PRECAUTIONS	6
5. INSPECTION ITEMS	7
6. LIMITED WARRANTY	7

1. SPECIFICATIONS

SEAT WIDTH:	19"
SEAT DEPTH:	16"
SEAT HEIGHT:	20"
FRAME TYPE:	Folding Aluminium
ARM STYLE:	Fixed
FOOTREST TYPE:	Swing-away
FOOT PLATES:	Flip up PVC
REAR TYRES:	24" x 1 1/4" Flat free
CASTERS:	8" x 1" Flat Free
WHEEL LOCKS:	Push to lock extended and Assistant hand brakes
MAX USER WEIGHT:	130 kg
PRODUCT WEIGHT:	14.6 kg

2. WHEELCHAIR COMPONENTS



- 1. Push handles
- 2. Back upholstery
- 3. Rear wheel
- 4. Pushrims
- 5. Cross-brace
- 6. Anti-tipper

- 7. Fork
- 8. Caster
- 9. Armrest
- 10. Brake handles
- 11. Footrest
- 12. Footplate

3. INSTRUCTIONS

3.1 Folding and Opening


- » To fold, flip up the foot-plates or remove the footrests altogether and pull up on the center front & back of seat upholstery.
- » To unfold, push down on the seat side frame until the seat locks into place. Put the footrests on and/or fold down the foot-plates.



3. INSTRUCTIONS

3.2 Adjusting the Footrest Length

Using the wrench provided, place it over the bolt and turn to left to loosen or to the right to tighten in place. Adjust the length by sliding the footrest up or down so that the person's full thigh is supported by the seat.

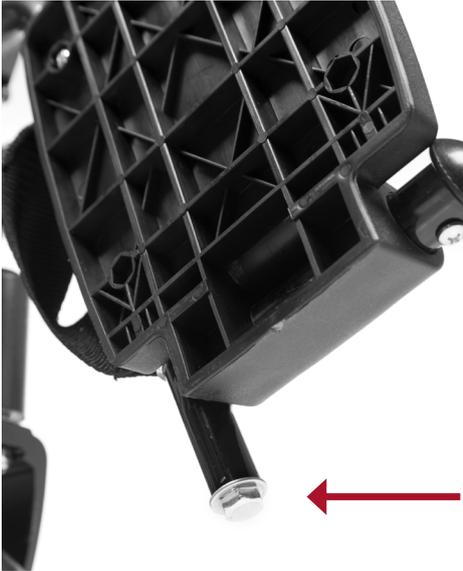

Remember

Foot-plates need to keep at least a 2" clearance from the ground to overcome thresholds



Remember

If the knees are too high the person may get a sore bottom from increased pressure. If the knees are too low the person may slide out of the chair

3. INSTRUCTIONS

3.3 Removing & Replacing Footrests

Swing the footrest to the side by pushing the lever **(1)** Grasp the top bar of the footrest and lift up **(2)**, releasing the footrests from the pins. Repeat on the other side. Holding the footrest in one hand, from the side align the pins with the holes in the footrest and attach **(3)**. Swing footrest back in place.



1



2



3

3.4 Applying the Brakes

Push the brake handle forward to lock and pull back to release.



Remember

To apply the brakes before getting in or out of the chair

The wheels and casters have maintenance free solid tyres installed that will not become flat if they are cut or punctured by debris on the ground.

The chair is fitted with 8" diameter casters and 24" diameter rear wheels to make travel over rough terrain and small obstacles easier.

On very rough or soft terrain it will be easier, and safer, for the care giver to pull the chair backwards until smoother or harder terrain is reached.

3. INSTRUCTIONS

3.5 Removing and replacing the quick release wheels



Push and hold the center of the hub (as shown in the image) and pull the wheel from the frame. To replace the wheel, simply push the wheel securely back onto the frame.

4. SAFETY PRECAUTIONS

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened
3. Ensure wheel locks are engaged before transferring to or from the wheelchair.
4. Use the supplied seat belt when being pushed by others.
5. **DO NOT** overload this wheelchair. The weight capacity is 300 pounds including the user and any things they may be carrying.

5. INSPECTION ITEMS

- » Ensure the chair rolls straight (no excessive drag or pull to one side).
- » Inspect the upholstery for tears and/or sagging.
- » Inspect hand grips for wear, looseness or damage.
- » Inspect for any loose or broken hardware or parts.
- » Inspect for cracked, broken or loose spokes.
- » Ensure the hand brakes DO NOT interfere with the tyres when rolling.
- » Ensure the hand brakes are easy to engage.

6. LIMITED WARRANTY

CareCo products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty.

CareCo reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

Vista8

CareCo